

# SLx 45

## Slit Lamp

# User Manual



SLx 45 is a registered Trade Mark of Labotech Microscopes India Pvt. Ltd.

Labomed is the registered Trade Mark of Labo America Inc.

All other trade marks are property of their respective owners.

The information contained in this document was accurate at the time of publication. Specification subject to change without notice.

Labotech and Labo America Inc. reserves the right to make change in the product described in this manual without notice and without incorporating those changes in any products already sold.

ISO 13485 Certified - LABOMED products are designed and manufactured under quality management system conforming to ISO 13485 requirements.

No parts of this publication may be reproduced, stored in the retrieval system, or transmitted in any form or by any means, electronic, mechanical, recording, or otherwise without the prior written permission of LABOTECH.

Caution: Federal law restrict this devices to sale by or on the order of a licensed physician. Rx only.

# TABLE OF CONTENTS

Warnings and Cautions	4
Symbol Information	6
Introduction	7
Indications for Use	7
Contraindications	7
Features & Functions	8
Parts Identification	8
SLx 45 Package Contents	8
Accessories	8
Setup	9
Unpacking & Installation	9
Application of Input Power	12
Disconnection of Input Power	12
Instructions for Use	13
Operation	13
Adjusting Slit Length	14
Filters	14
Slit Rotation	15
Illumination Inclination	15
Slit Centration	15
Cleaning & Maintenance	16
Cleaning	16
External Cleaning	16
Forehead / Chin rest Preparation	16
Cleaning the Glide Plate	16
Mirror Cleaning / Replacement	16
Cleaning and Disinfection	17
Changing the Halogen Bulb	18
Replacement of Chin Rest Light Indicator	18
Changing the LED	19
Fuse Replacement	20
Troubleshooting	21
Specifications	22
Physical Dimensions	22
Electrical	22
Operational Conditions	22
Optics	22
Movement Ranges	23
Disposal	23
Software Revision	23
System Classification	23
Restocking chinrest tissue	23
System Configuration	24
Guidance Tables	25
Warranty	29

# WARNINGS AND CAUTIONS

LABOMED is not responsible for the safety and reliability of this instrument when:

- Assembly, disassembly, repair, or modification is made by unauthorized dealers or persons.
- The instrument is not used in accordance with this user manual.

**A WARNING is an instruction that draws attention to the risk of injury or death.**



**WARNING:** UNITED STATES FEDERAL LAW AND EUROPEAN REGULATIONS REQUIRE THAT THIS DEVICE BE PURCHASED ONLY BY A PHYSICIAN OR A PERSON ACTING ON BEHALF OF A PHYSICIAN.

**WARNING:** THIS INSTRUMENT SHOULD BE USED IN STRICT ACCORDANCE WITH THE INSTRUCTIONS OUTLINES IN THIS USER'S GUIDE. THE SAFETY OF THE OPERATOR AND THE PERFORMANCE OF THE INSTRUMENT CANNOT BE GUARANTEED IF USED IN A MANNER NOT SPECIFIED BY LABOMED TECHNOLOGIES.

**WARNING:** DO NOT REPAIR OR SERVICE THIS INSTRUMENT WITHOUT AUTHORIZATION FROM THE MANUFACTURER. ANY REPAIR OR SERVICE TO THIS INSTRUMENT MUST BE PERFORMED BY EXPERIENCED PERSONAL OR DEALERS WHO ARE TRAINED BY LABOMED OR SERIOUS INJURY TO THE OPERATOR OR PATIENT MAY OCCUR.

**WARNING:** MODIFICATIONS TO THIS INSTRUMENT ARE NOT ALLOWED. ANY MODIFICATION TO THIS UNIT BE AUTHORIZED BY LABOMED OR SERIOUS INJURY TO THE OPERATOR OR PATIENT MAY OCCUR.

**WARNING:** IF THIS INSTRUMENT IS MODIFIED, APPROPRIATE INSPECTION AND TESTING MUST BE CONDUCTED TO ENSURE CONTINUED SAFE USE OF THIS INSTRUMENT.

**WARNING:** TO AVOID RISK OF ELECTRIC SHOCK, THIS EQUIPMENT MUST ONLY BE CONNECTED TO A SUPPLY MAIN WITH PROTECTIVE EARTH OR DAMAGE TO THIS INSTRUMENT AND/OR INJURY TO THE OPERATOR OR PATIENT MAY OCCUR.

**WARNING:** ENSURE THAT THE VOLTAGE APPLIED TO THE UNIT IS THE SAME AS THE VOLTAGE THAT IS INDICATED ON THE DATA PLATE OR DAMAGE TO THE UNIT MAY OCCUR.

**WARNING:** THIS INSTRUMENT MUST BE PLUGGED INTO AN OUTLET WITH AN EARTH GROUND. DO NOT REMOVE OR DEFEAT THE EARTH GROUND CONNECTION ON POWER INPUT CONNECTOR OR THE UNIT'S POWER CORD OF THIS INSTRUMENT OR DAMAGE TO IT AND/OR INJURY TO THE OPERATOR OR PATIENT MAY OCCUR.

**WARNING:** THE EQUIPMENT OR SYSTEM SHOULD NOT BE USED ADJACENT TO OR STACKED WITH OTHER EQUIPMENT AND THAT IF ADJACENT OR STACKED USE IS NECESSARY, THE EQUIPMENT OR SYSTEM SHOULD BE OBSERVED TO VERIFY NORMAL OPERATION IN THE CONFIGURATION IN WHICH IT WILL BE USED.

**WARNING:** THIS INSTRUMENT IS NOT SUITABLE FOR USE IN THE PRESENCE OF FLAMMABLE ANESTHETIC MIXTURES, SUCH AS OXYGEN OR NITROUS OXIDE.

**WARNING:** BECAUSE PROLONGED INTENSE LIGHT EXPOSURE CAN DAMAGE THE RETINA, THE USE OF THE DEVICE FOR OCULAR EXAMINATION SHOULD NOT BE UNNECESSARILY PROLONGED, AND THE BRIGHTNESS SETTING SHOULD NOT EXCEED WHAT IS NEEDED TO PROVIDE CLEAR VISUALIZATION OF THE TARGET STRUCTURES. THIS DEVICE PROVIDED WITH FILTERS THAT ELIMINATE UV RADIATION (<400NM) AND, WHENEVER POSSIBLE, FILTERS SHORT- WAVELENGTH BLUE LIGHT (<420NM).

**WARNING:** THE USE OF ACCESSORIES OR CABLES OTHER THAN THOSE SPECIFIED, WITH THE EXCEPTION OF THOSE SOLD BY THE MANUFACTURER AS REPLACEMENT PARTS FOR THE INTERNAL COMPONENTS, MAY RESULT IN INCREASED EMISSIONS OR DECREASED IMMUNITY OF THE EQUIPMENT OR SYSTEM.



## WARNINGS AND CAUTIONS (continued)

A **CAUTION** is an instruction that draws attention to the risk of damage to the product.



**CAUTION:** THE INTERNAL CIRCUITRY OF THE INSTRUMENT CONTAIN ELECTROSTATIC SENSITIVE DEVICES (ESD) THAT MAY BE SENSITIVE TO STATIC CHARGES PRODUCED BY THE HUMAN BODY. DO NOT REMOVE THE COVERS WITHOUT TAKING PROPER ESD PRECAUTIONS.

**CAUTION:** DO NOT USE SOLVENTS OR STRONG CLEANING SOLUTIONS ON ANY PART OF THIS INSTRUMENT, AS DAMAGE TO THE UNIT MAY OCCUR SEE THE CARE AND MAINTENANCE SECTION FOR DETAILED CLEANING INSTRUCTIONS.

**CAUTION:** MEDICAL ELECTRONIC EQUIPMENT NEEDS SPECIAL PRECAUTIONS WITH RESPECT TO ELECTROMAGNETIC CHARGE (EMC) AND NEEDS TO BE INSTALLED AND SERVICED ACCORDING TO THE EMC INFORMATION PROVIDED IN THE ACCOMPANYING DOCUMENTS.

**CAUTION:** PORTABLE AND MOBILE RF COMMUNICATIONS EQUIPMENT CAN AFFECT MEDICAL ELECTRICAL EQUIPMENT.

**CAUTION:** THIS INSTRUMENT IS NOT TO BE USED NEAR HIGH-FREQUENCY EMITTING SURGICAL EQUIPMENT.

**CAUTION:** THIS INSTRUMENT IS NOT INTENDED TO BE CONNECTED TO EQUIPMENT OUTSIDE THE CONTROL OF LABOMED OR MUST BE TESTED TO AN APPLICABLE IEC OR ISO STANDARDS.

**CAUTION:** CHIN REST PAPER-FOR SINGLE USE ONLY. REPLACE WITH EACH PATIENT.

# EXPLANATION OF SYMBOLS

The following symbols appear on the instrument:



Caution symbol indicating important operating and maintenance instructions that are included in this User's Guide.



Type B Applied Part



Alternating Current Power



Protective Earth



Connection ON/OFF



Manufacture

REF

Catalog Number

S/N

Serial Number



Waste of Electrical and Electronic Equipment



Compliance to medical Device Regulation EU MDR 2017/745



Accompanying Document must be consulted.



Authorized Representative in European Community.



Fragile Contents in Shipping Container- Handle with care



Keep Dry- Package shall be kept away from rain.



This way Up- Indicates correct upright position of package.

# INTRODUCTION

Congratulations on your purchase of the SLx 45 Slit Lamp.

This User Manual is designed as a training and reference manual for the operation and maintenance of the instrument. We recommend that you read it carefully prior to use and follow the instructions to ensure optimum performance of your new instrument. Properly trained eye care professionals such as ophthalmologists, optometrists, opticians and eye care technicians should operate this instrument.

Please retain this manual for future reference and to share with other users. Additional copies can be obtained from your authorized LABOMED dealer or from the LABOMED Customer Service Department at:

Tel: (510) 445-1257  
Fax: (510) 991-9862  
Email: sales@laboamerica.com

## INDICATIONS FOR USE

The SLx 45 Slit Lamp is an AC-powered slit lamp biomicroscope that is intended for use in examining the anterior segment, from the corneal epithelium to the posterior capsule. It is used to aid in the diagnosis of diseases or trauma, which affect the structural properties of the anterior segment of the eye.

## CONTRAINDICATIONS

None.

# FEATURES AND FUNCTIONS

## PART IDENTIFICATION

1. On/Off Switch
2. Illumination level Control
3. Guide Plate
4. Joystick for horizontal and vertical movement
5. Microscope Arm Lock-Knob
6. Slit Width Knob
7. Inclination Latch Release
8. Centering Knob
9. Breath Shield Mount
10. Eyepieces
11. Focusing Ring
12. Magnification Dial
13. Lamp Housing Cover
14. Goose Neck LED indicator
15. Filter Lever
16. Slit Rotation/ Length Knob
17. Slit Rotation Scale
18. Microscope Lock Knob
19. Illumination Arm Lock Knob
20. Instrument Base Lock Knob
21. Guide Rails Covers
22. Patient Handles
23. Binocular Head Assembly
24. For Head Rest Assembly
25. Chin Support Assembly
26. Chin Rest Assembly
27. Metal Plate Frame
28. Magnichanger Assembly
29. CMO
30. Aperture Size Display
31. Power Suply Box

## SLx 45 PACKAGE CONTENTS


1. SLx 45 Slit Lamp HL (8127400)
2. SLx 45 Slit Lamp LED (8127401)
3. User's Guide (8127000-795)

## ACCESSORIES

1. Focusing Rod Assembly (15120-226)
2. Hex Wrench - 3mm (LK-003)- X54264
3. Hex Wrench - 4mm (LK-004)- X54248
4. Hex Wrench - 5mm (LK-005)- X54318
5. Blue Dust Cover (15140-004)
6. Halogen lamp,Main (15121)
7. Guide Rail Covers (15140-003)
8. Replacement Fuses (8124900-900)
9. Chin Rest Paper-1 pack (15140-006)

## LED ASSEMBLY

1. Heat Sink (8127201-101)
2. LED 5 WATT (8134300-904)

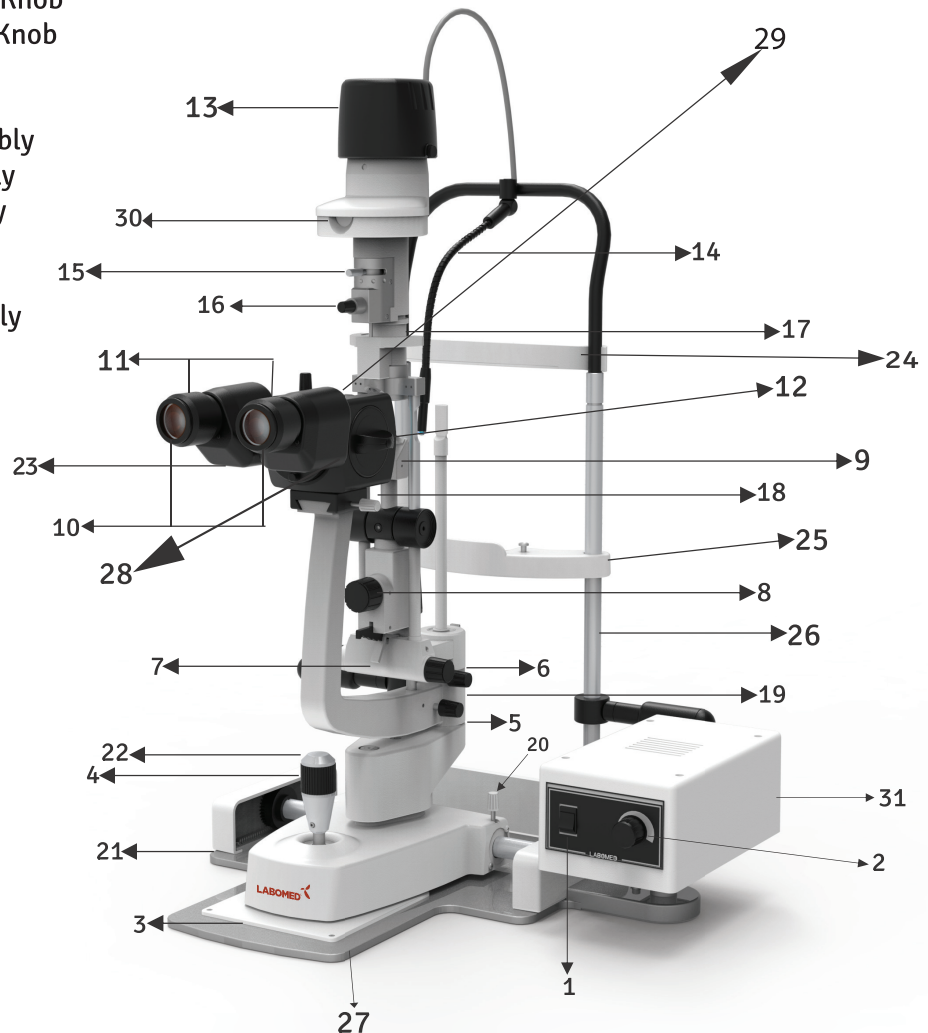

# SETUP

## UNPACKING AND INSTALLATION

1. Open the outside shipping box and remove the four (4) inner boxes.
2. Remove the User's Guide and follow it for Slit lamp installation.
3. Open the box marked as "Platform Assembly" and take out the table top and power supply bracket (shown as "A") in fig. 1.
4. Open the box marked as "Chin Rest Assembly" and take out the complete chin rest assembly as shown in fig. 2.
5. Assemble chin rest assembly to table top using (6) Allen Screw of M3, securing with help of Allen Wrench 2.5mm. refer in fig. 3.
6. Assemble the power supply bracket to the table top using (2) M4 Allen screws provided with it. referred as (B) in fig. 4.



Figure.1 Table Top & Power supply bracket



Figure. 2 Chin Rest Assembly



Figure. 3 Chin Rest attaching Screw

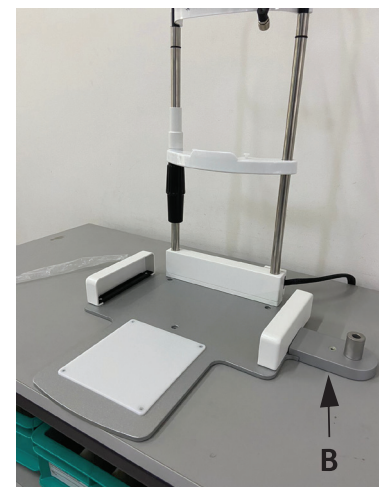


Figure. 4 Chin Rest Assembly

# SETUP (continued)

## UNPACKING AND INSTALLATION (continued)

7. Referring fig.-5. Open the box with the Microscope Assembly. Remove the Microscope Assembly, Base Assembly, Power Supply and Accessories.

8. Install the Base Assembly onto the tracks of the table top and slide the guide rail covers around the tracks. refer to fig.-6.

9. Referring fig.-6a. Open the box marked as "Illumination and Microscope Arm Assembly". Take out the Illumination and Microscope Arm Assembly.

10. Assembly illumination tower to the Base by unscrewing the M8 screw from its bottom using 5.0mm Allen Key. Allen Key is provided with tool kit. Refer fig.-6b.

**NOTE:** The Illumination and Microscope Arm Assembly are connected as one piece. Refer fig.- 6b.



Figure.5 Microscope, Base, Power Supply & Accessories

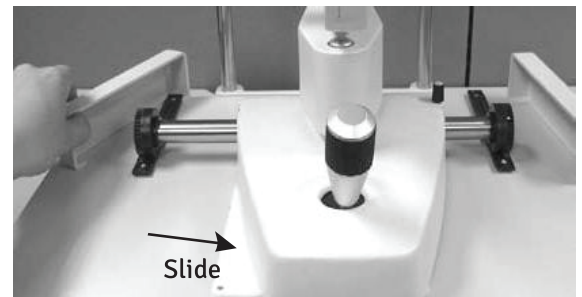


Figure. 6 Install Base Assembly



Figure.6a Illumination & Microscope Arm Assembly



Figure. 6b Mounting Illumination Tower & Arm



# SETUP (continued)

## UNPACKING AND INSTALLATION (continued)

11. Assemble Power Supply to the table top just by pushing into the hole shown as (D), Refer fig.-7.

12. Connect the earthing wire with power supply at point "E" using 3mm Allen Wrench. Refer fig.-8b.

13. Attach the Connector of chin rest target light indicator to the back of power supply. Refer figure-8.

14. Connect the Lamp Assembly to the Head Rest Assembly by connecting the Lamp Assembly Harness to the lamp Assembly. Refer to figure-8.

NOTE: Line up the prongs in the Lamp Assembly with the holes in the Harness when connecting. Secure it by turning the metal collar clockwise. Refer to figure-8a.

15. Connect the Lamp Assembly Plug to the Power Supply Assembly by plugging it in, then turning the plastic collar clockwise, securing the plug. Refer to figure-8b.

16. Connect the chin rest target light connector to the Power Supply Assembly by plugging it in. Refer to figure-8b.

17. Using the 4mm Hex Wrench, adjust the Patient Handles by loosening the Allen Cap Screws that are securing them to the Chin rest Posts. Slide the Patient Handles up or down to the desired height, and secure them in place by tightening the Allen Cap Screws. Refer figure 9.

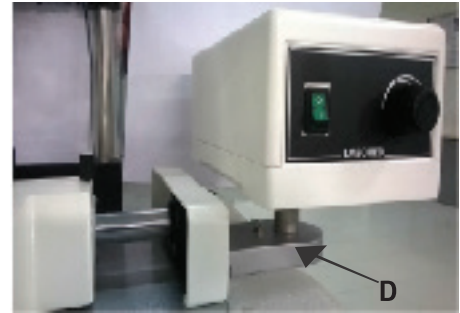


Figure.7 Assembling Power Supply on the Power Supply Bracket

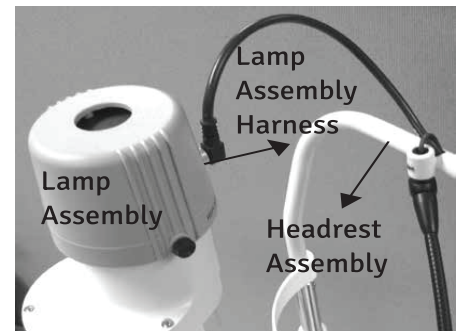


Figure.8 Harness Connected

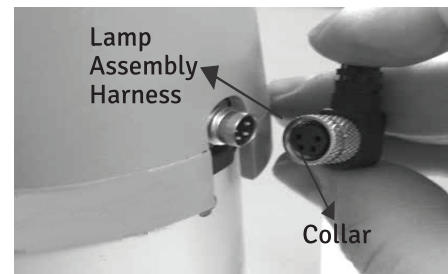


Figure.8a Harness Connections


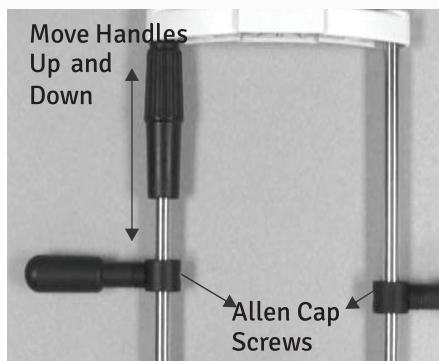


Figure. 9 Adjusting Patient handle Height

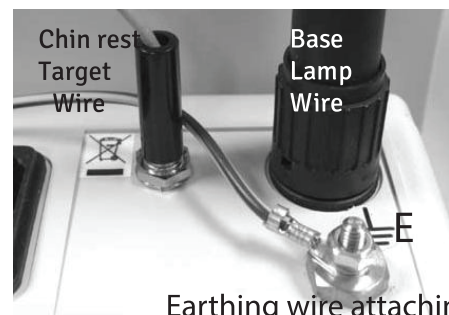

Earthing wire attaching  
Figure. 8b Connections

# SETUP (continued)

## UNPACKING AND INSTALLATION (continued)

18. Install the Microscope Assembly onto the top of the Arm by sliding it into position, making sure it is up against the stop. Then, tighten the lock Knob located on the right side of the Microscope Assembly. Refer to figure-10.

**NOTE:** Do not adjust the Microscope stop knob behind the base of the microscope, or the vertex distance will cause misalignment of focus and require re-calibration of the slit lamp assembly.

19. Remove the accessories and store them in an appropriate place so that when they are needed they will be available. Refer to figure-11.

## APPLICATION OF INPUT POWER

**WARNING:** CARE MUST BE TAKEN TO ARRANGE THE CABLES FOR THE ACCESSORIES SUCH THAT THEY DO NOT PRESENT A TRIPPING HAZARD TO THE EXAMINER OR A DANGER TO THE PATIENT.

**WARNING:** POSITION THIS INSTRUMENT SO THAT IT IS NOT DIFFICULT TO OPERATE THE DISCONNECTION DEVICE (PLUG).

1. After the unit is in its secure location, apply the correct input voltage to the instrument using the Power Cord from the Accessory Tray.

**NOTE:** The power inlet is located on the backside of the Power Supply Assembly.

2. Press down on the “I” located on the ON/OFF Switch. Refer to figure-12.

**NOTE:** The ON/OFF Switch will illuminate green when there is power to the unit. When the ON/OFF Switch is set to off, the green light will turn off.

## DISCONNECTION OF INPUT POWER

1. At any time, the power switch can be set to OFF. The unit does not have a power down sequence. To terminate operation of this instrument, press the ON/OFF switch to the OFF position (O).

2. If this instrument is intended to be OFF for an extended period of time, it can be disconnected from power by detaching the power cord from the receptacle.

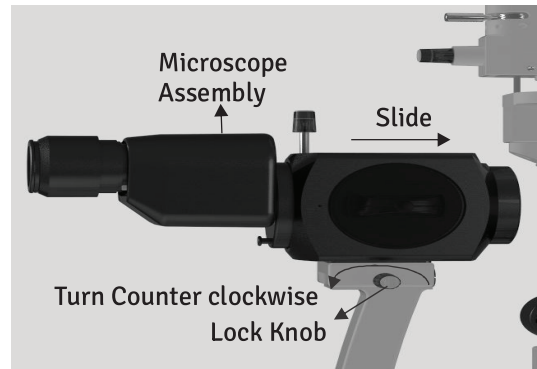


Figure. 10 Microscope Install

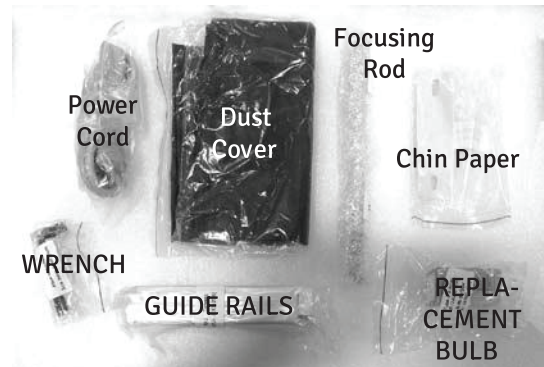


Figure. 11 Accessories



Figure. 12 Power Supply Assembly



# INSTRUCTIONS FOR USE

## OPERATION

1. Turn on the power using the On/Off switch located on the front of the Power Supply Assembly. Brightness can be adjusted by rotating the illumination level knob.

**NOTE:** The maximum position is for intermittent use only. Continuous use will shorten the lamp/ LED life.

2. Insert the Focusing Rod in the pivot post of the instrument body to make rough PD and focus adjustments. Refer to figure IN-1.

3. Position the light onto the flat surface of the focusing rod and adjust the pupillary distance and focus of the eyepieces to suit the needs of the operator.

4. Using the Slit Width Knobs, adjust the projected slit so that the thinnest slit is shown on the Focusing Rod. Refer to figure IN-2.

**NOTE:** The thinnest line will allow for greater accuracy.

5. Remove the Focusing Rod.

6. To position a patient, adjust the chin rest height by turning the Chin rest Elevation Handle on the post of the Chin Rest Assembly until the patient's canthus is in line with the canthus mark on the chin rest post. Refer to figure IN-2 & IN-3.

7. Microscope elevation is adjusted by rotating the joystick and observing the slit image through the Microscope Assembly until slit is centered on the patient's cornea. Refer to figure IN-4.

8. Move the slit lamp with the joystick held firmly and slightly angled toward the patient, until the slit appears sharply on the cornea.

**NOTE:** The accuracy of this rough adjustment should be checked by the naked eye. The fine adjustment is performed while observing the slit through the microscope.

9. Tilt the Joystick, which is now held lightly at its upper end, until the slit appears sharply at the depth of the eye which is to be observed.

10. The horizontal motion of the base can be locked by tightening the Base Locking Screw. Refer to figure IN-4.

**NOTE:** Lock the base whenever the lamp is not in use.

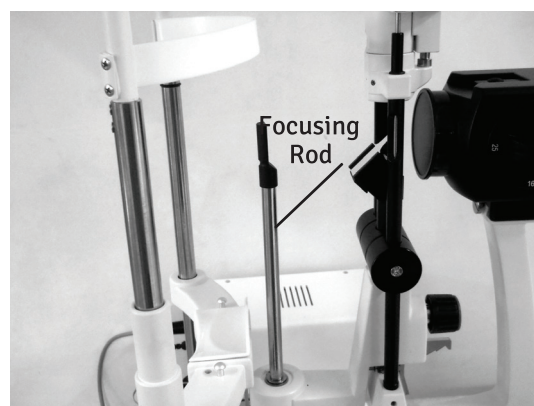


Figure IN-1 Install Focusing Rod

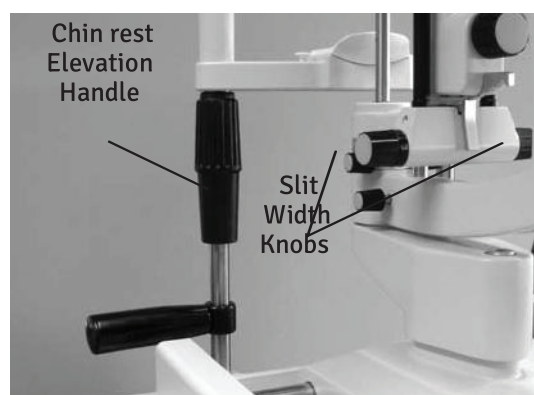


Figure IN-2 Adjust Patient Height

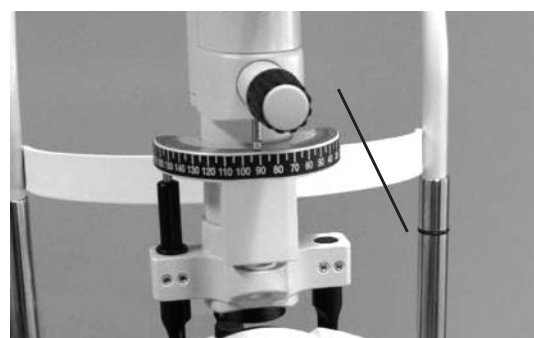


Figure IN-3 Adjust Patient Height

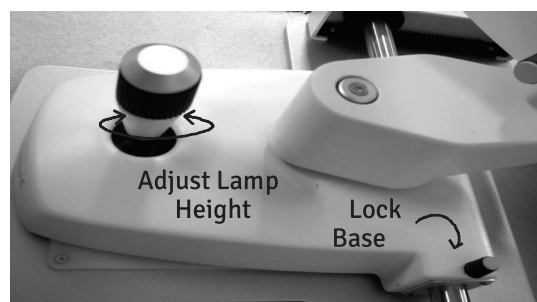


Figure IN-4 Adjust Height

# INSTRUCTIONS FOR USE (continued)

## OPERATION (continued)

11. The slit width can be adjusted by rotating the Slit width Knobs. Refer to figure IN-2.

12. The angle between the illumination system and the microscope can be varied between 0° and 90° to either the left or right. Refer to figure IN-5.

13. The illumination angle is indicated on the Illumination Angle Scale on the slit lamp arm. Refer to figure IN-6.

14. Magnification is altered by rotating the Magnification Dial on the Microscope Assembly. Refer to figure IN-5.

**NOTE:** The magnification of each click-stop position is engraved on the Magnification Dial.

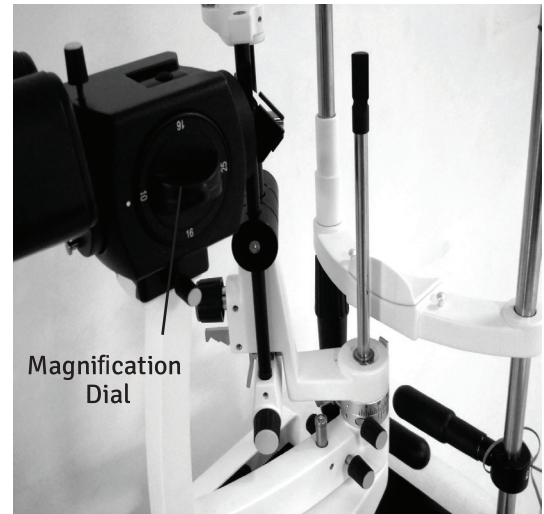


Figure IN-5 Illumination Angle

## ADJUSTING SLIT LENGTH

The slit length is adjusted by rotating the Slit length/Rotation Dial. The dial has five steps for adjustments. They are 12, 9, 5, 3, 1 and 0.3mm diameter and Continuous length. Refer to figure IN-7.

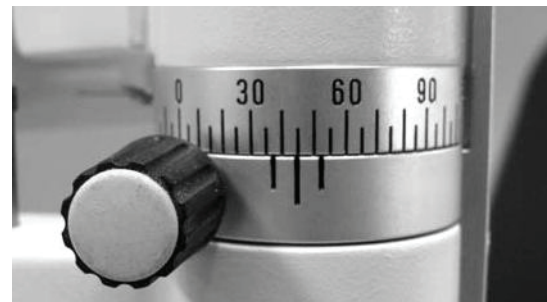


Figure IN-6 Illumination Angle Scale

## FILTERS

There are five filters that can be used by indexing the filter lever to the coordinating Filter Dot. The Filter Dots are color coded. Refer to figure IN-8. The color coded filter are as follows:

- Blue dot = Cobalt Blue
- Red dot = Open
- Yellow dot = Heat Absorbing
- Gray dot = Neutral Density
- Green dot = Red-free

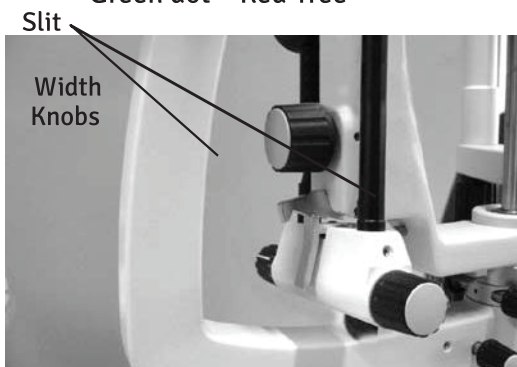


Figure IN-7 Slit Width Knob

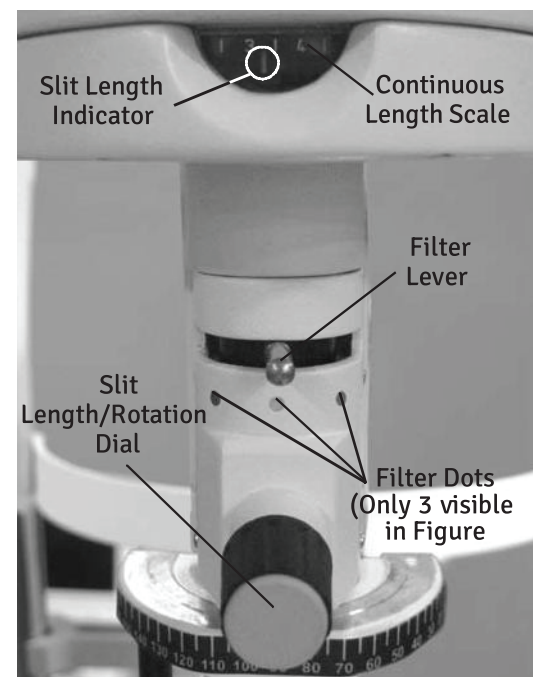


Figure IN-8 Slit Width and Filters

# INSTRUCTIONS FOR USE (continued)

## OPERATION (continued)

### SLIT ROTATION

By grasping the Slit Rotation Knob, the lamp housing can be rotated. This, in turn, rotates the slit from vertical to horizontal in either direction. The slit positions are click-stopped in 45° increments and stopped at 0° and 180° and is indicated by the scale. Refer to Figure IN-9.

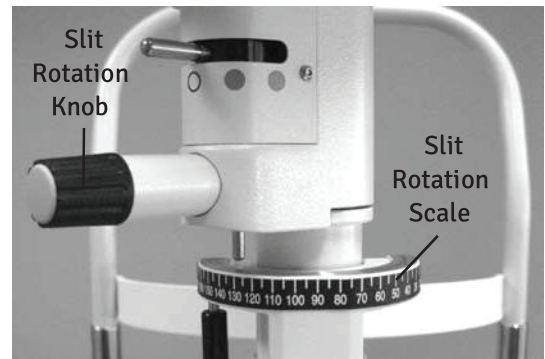


Figure IN-9 Slit Rotation Scale

### ILLUMINATION INCLINATION

The Illumination Assembly can be inclined in the horizontal plane in 5° steps for a total of 20°. Tilt the Illumination Assembly by depressing the inclination Latch Release and pulling the base of the Illumination Assembly toward the operator. Refer to figure IN-10.

### SLIT CENTRATION

When the centering screw is loosened, the slit can be scanned away from the center of the field of vision for retro-illumination, scaler scatter, etc. The slit image is centered again by tightening the screw. Refer to figure IN-11.

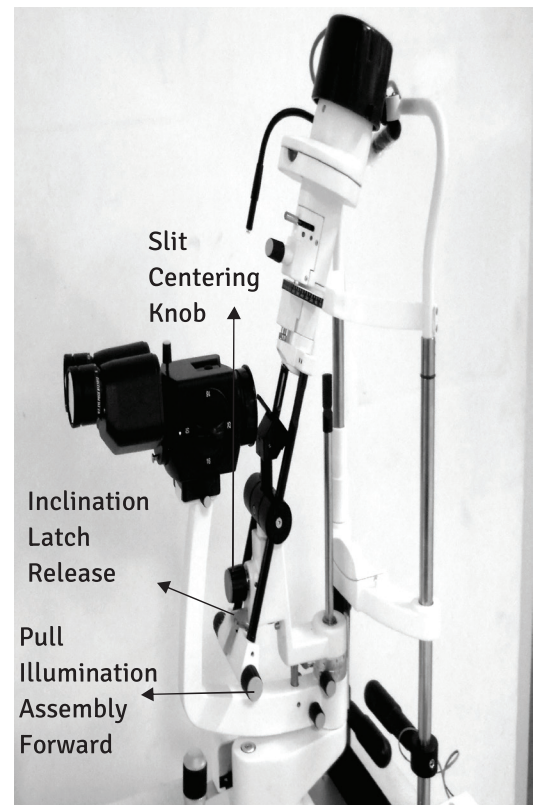


Figure IN-10 Illumination Inclination

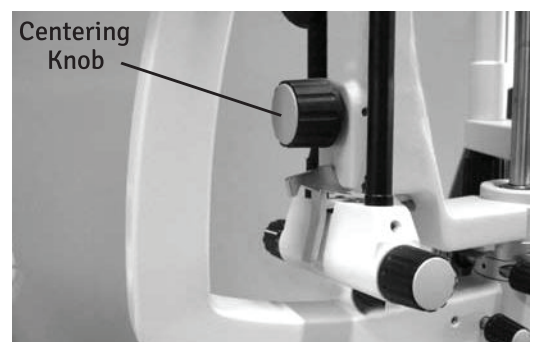


Figure IN-11 Slit Centering



# CLEANING & MAINTENANCE

**WARNING: RISK OF ELECTRIC SHOCK. ALWAYS DISCONNECT THE POWER CORD FROM THE WALL AND THE INSTRUMENT BEFORE PERFORMING ANY OF THE FOLLOWING CARE AND MAINTENANCE PROCEDURES.**

## CLEANING

### EXTERNAL CLEANING

Clean the external surfaces of this instrument using a clean, soft cloth moistened with a mild detergent solution of liquid dish soap (filtered below 5 microns). Refer to figure MM-1.

### FOREHEAD/CHIN REST PREPARATION

For hygienic reasons, wipe the forehead rest with an alcohol wipe and change the chin rest papers after each patient.

### CLEANING THE GUIDE PLATE

If the Guide Plate is dirty it may cause a rough feeling when maneuvering the base of the slit lamp. Clean the Guide Plate with a soft cloth lightly dampened with a mild soap and water solution.

### MIRROR CLEANING / REPLACEMENT

When cleaning the mirror, clean off the mirror using dry air, then gently wipe with a soft lint-free cloth. If replacing the mirror, grasp the narrow shank of the mirror and pull upwards. Replace it with a new one by sliding it in place. Refer to figure MM-02.

### USER ENVIRONMENT REQUIREMENT

- Maintain Operational Environment as specified in Specification section of this manual.
- Do not use this Instrument in corrosive environment to prevent rusting etc.

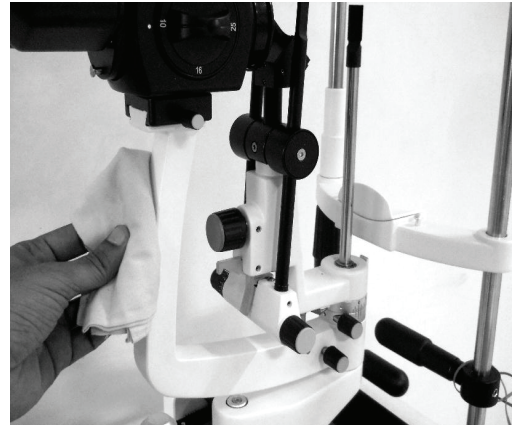


Figure CM-1 Cleaning Main Unit

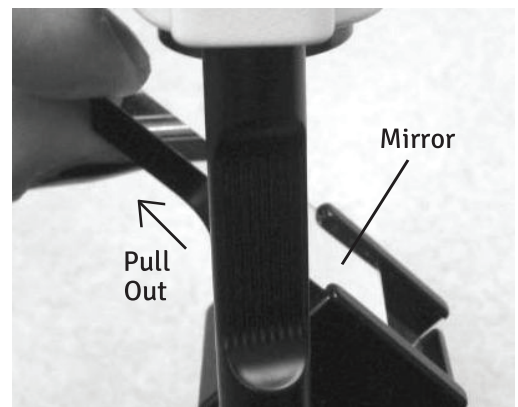


Figure CM-02 Remove Mirror

# CLEANING & DISINFECTION

**Following disinfectants are recommended for cleaning and disinfection.**

1. Normal household bleach (Sodium hypochlorite 5%)- strength 5000 ppm(10 parts water 1 part bleach).
2. 70% Isopropyl alcohol.

**Procedure:**

- Take a muslin cloth.
- Moist it to feel wet
- Disinfect/clean the surface gently.

**Note – 1. Use mask and Gloves while performing cleaning and disinfection.**

2. **While cleaning, muslin cloth should not be dripped wet to prevent seepage and rusting to running/bare parts.**
3. **Alcohol is flammable,its use as a surface disinfectant should be in well-ventilated spaces only.**

## CHANGING THE HALOGEN BULB

(For HL illumination Assembly)

**WARNING:** NEVER REMOVE A BULB THAT HAS RECENTLY BEEN IN USE AS IT WILL BE VERY HOT. WAIT UNTIL IT HAS COOLED AND USE GLOVES OR A THICK CLOTH WHEN HANDLING ANY HALOGEN BULB.

**WARNING:** NEVER TOUCH A HALOGEN BULB WITH BARE HANDS AS FINGERPRINTS WILL SHORTEN THE BULB LIFE.

1. Remove input power to the instrument.
2. Remove the Lamp House Cover by loosening the Two Screws securing the Cover. Refer to figure MM-03.
3. Unscrew the Screw securing the Metal Tab holding the Bulb Holder in place. Refer to figure MM-04
4. Gently pull out the Bulb Holder and Bulb. Refer to figure MM-05.
5. Grip the Bulb by the Metal Disk that is attached to it, and pull it out of the Bulb Holder. Refer to figure MM-06.
6. Replace the Bulb with a new one and install it by pushing the prongs into the Bulb Holder so that the Notch is positioned to the right. Refer to figure MM-06 and MM-07.

**NOTE:** There is a Cut Out in the metal Disk on the Bulb. Ensure that the Bulb is placed properly so that the Cut Out fits into the Notch. Refer to figure MM-07.

7. Secure the Bulb with the Metal Tab and Screws. Refer to figure MM-04.
8. Install the lamp Housing Cover the secure it by tightening the Two Screws. Refer to Figure MM-03

## Replacement of Chin Rest Light Indicator

1. Unscrew the LED cap refer fig. CM-08.
2. Pull out the defective LED and replace with new LED. refer fig. CM-08a. Thread in the LED cap back.

**NOTE:** If New LED does not light up after switching on. remove it and re-fix after changing its polarity.

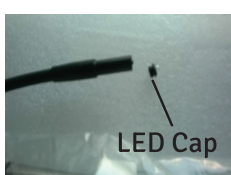


Figure CM-08 Changing Target Light LED

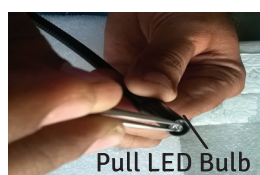


Figure CM-08a Changing Target Light LED

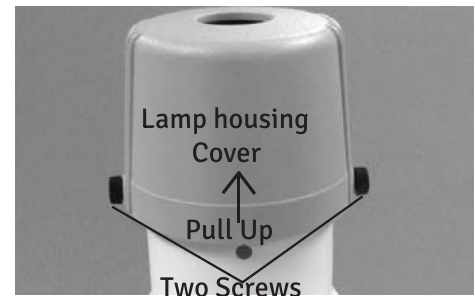


Figure CM-03 Remove cover

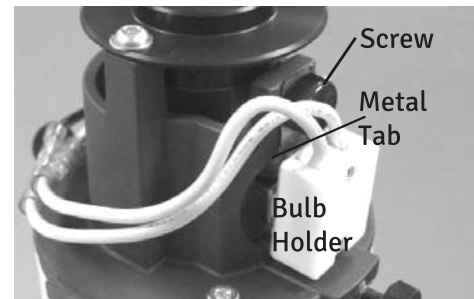


Figure CM-04 Securing Screw

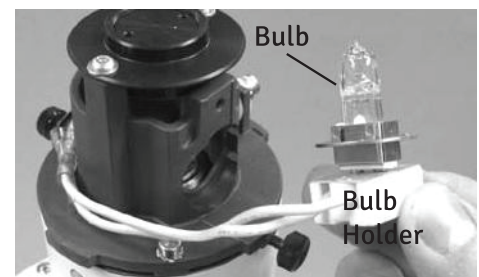


Figure CM-05 Bulb

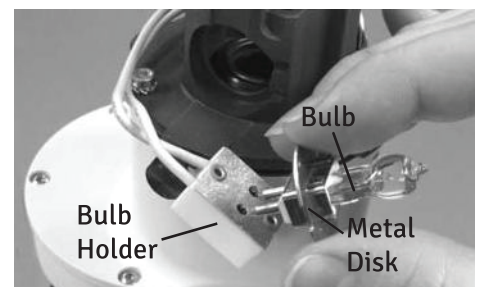


Figure CM-06 New Bulb

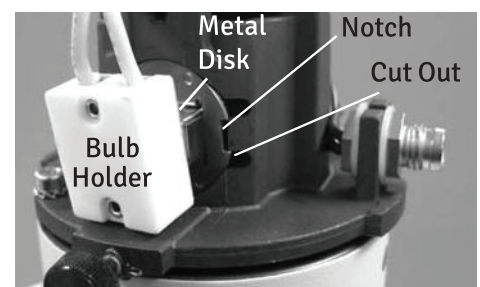


Figure CM-07 Notch

## CHANGING THE LED

(For LED illumination only)

**WARNING:** NEVER REMOVE A LED THAT HAS RECENTLY BEEN IN USE AS IT MAY BE VERY HOT WAIT UNTIL IT COOLED.

**WARNING:** NEVER TOUCH A LED WITH BARE HANDS AS FINGERPRINT WILL SHORTEN THE LED LIFE.

1. Remove input power to the instrument.
2. Remove the lamp Housing Cover by loosening the Two Screws securing the Cover and lifting it straight off. Refer to figure CM-03.
3. Unscrew the Two Screws to disassemble Heat Sink from the casting. Refer to figure CM-09.
4. Unscrew the LED from Heat Sink by unscrewing the Two Screws. Refer to figure CM-10.
5. Replace New LED and follow Reverse to complete assembly.



Figure CM-09



Figure CM-10

## FUSE REPLACEMENT

Replace the fuses in the Power Input Module with the fuses indicated in the **Specifications** section of this manual.

1. Remove input power to the instrument.
2. Press down on the top tab in the middle of the Power Input Module to release the Fuse Holder, and gently pull out the Fuse Holder by gripping the two small tabs. Refer to figures- MM-8 & MM-9.
3. Open the Door to the Fuse Holder by pulling it down. Refer to figure-MM-9.

**NOTE:** The Fuses will pop up when the door is open, making removal easier.

4. Install new fuses into the Fuse Holder that is indicated in the Specification section of this manual.
5. Install the Fuse Holder by closing the door, and pushing the Fuse Holder back until it snaps into place.

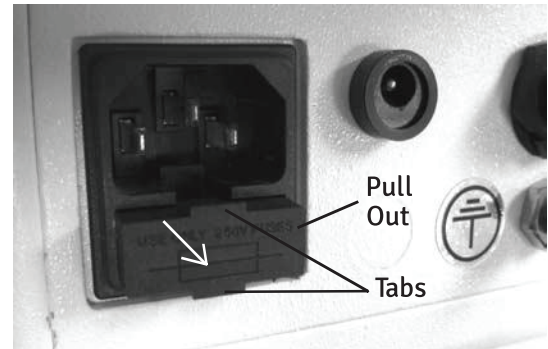


Figure MM-11 Pull Out

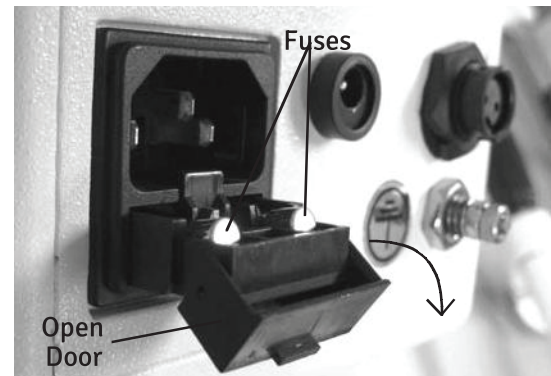


Figure MM- 12 Open Fuse Door



# TROUBLESHOOTING

The following chart outlines some common issues with the SLx 45 Slit Lamp and some steps you can take to correct the issue. If problems persist, please contact the LABOMED as listed in the Introduction section of this manual.

## Chart of Common Errors




ISSUE	PROBABLE CAUSE	POSSIBLE SOLUTION
Lamp won't turn on.	Incorrect input power supplied to the SLx 45 Slit Lamp.	Check the outlet to ensure proper power is being supplied.
	Defective Power Cord.	Replace the Power Cord.
	Defective Power Supply.	Replace the Power Supply.
Slit Lamp won't move.	Rubber stopper may be attached under the joystick.	Remove the rubber stopper.
	Base Lock Screw may be tightened.	Loosen the Base Lock Screw.
Rough base movement.	Rubber stopper may be attached under the joystick.	Remove the rubber stopper.
	Bearings may be damaged.	Replace the base.
	Shaft may be damaged.	Replace the base.
Fixation light does not light up.	Fixation Light Harness not plugged into the Power Supply Assembly.	Ensure the Fixation Light Harness is properly seated in the Power Supply Assembly.
	Defective Power Supply.	Replace the Power Supply.
Light too dim.	Incorrect wattage for bulb being used.	Replace with the proper Bulb.
	Bulb/LED not installed properly.	Check bulb/LED and ensure notch lines up with bulb/LED housing.
Double slit visible in microscope.	Microscope not focused on focusing rod before use.	Install focusing rod and check to ensure microscope is focused on it.
	Bulb/LED not installed properly.	Check bulb/LED and ensure notch lines up with bulb housing.

The following is a checklist of items that need to be assessed in order to determine if the SLx 45 Slit Lamp requires servicing.

- Check the outside of the Slit Lamp for any damage or missing components.
- Inspect the power cord for damage.
- Test the lamp by turning the lamp on and turning the light all the way to its brightest setting, and all the way down to its lowest setting.
- Check to ensure all switches are functioning properly.
- Check the filters by cycling through all the options.
- Check the Slit Wheel by cycling through all the options.
- Check the base movement.

# SPECIFICATIONS

Catalog Number 8127000-795

<b>Physical Dimensions</b>	
Size:	Weight, unpacked: 39.35 lbs. (17.85 Kg)
Height: 26.8 in. (68.0 cm)	Weight, packed: 52 lbs (23.64 Kg)
Width: 17.3 in. (44.0 cm)	
Depth: 15.3 in. (38.8 cm)	
<b>Electrical</b>	
voltage: 100-240 vAC	
Power consumption (max): 50-75VA	
Frequency: 50/60 Hz	
Fuses: T2.5A H 250v, 5x20mm,	
Halogen Bulb: (6v, 20W)/ LED 5W.	
<b>Operational Conditions</b>	
<b>Environmental</b>	
The environmental conditions are as follows:	
<b>Operating:</b>	
Temperature: 10° C (50° F) to 35° C (95° F)	
Relative Humidity: 30% to 75%	
Atmospheric Pressure: 80 kPa (23.6 in. Hg) to 106 kPa (31.3 in. Hg)	
<b>Transportation &amp; Storage:</b>	
Temperature -20° C (-4° F) to +70° C (158° F).	
Relative Humidity: 10% to 80% (non-condensing)	
Atmospheric Pressure: 50 kPa (14.8 in. Hg) to 106 kPa (31.3 in. Hg)	
Exposure to extreme temperature conditions indicated above must not exceed 15 weeks.	
<b>Optics</b>	
<b>Microscope</b>	Galilean
<b>Mag Change</b>	3 Step Drum Rotation/5 step optional
<b>Eyepiece</b>	12.5X
<b>Mag Ratio</b>	10X, 16X, 25X / 6.5X, 10X, 16X 25X, 40X
<b>Field of view</b>	24.0,15.0,9.5 / 37.0,24.0,15.0,9.5,6.0
<b>PD Range</b>	49 - 78 mm
<b>Diopter Adjustment</b>	+/- 8mm
<b>Slit Illumination</b>	6v 20W Halogen/ 5W LED
<b>Slit Width</b>	0 - 12 mm
<b>Slit Length</b>	1.6 - 12 mm
<b>Slit Apertures</b>	0.3, 1, 3, 5, 9, 12 mm
<b>Slit Rotation</b>	0° - 180°
<b>Slit Inclines</b>	5, -10, -15, -20 degrees
<b>Filters</b>	Red Free, Heat Absorbing, Cobalt Blue, Neutral Density
<b>Field of View</b>	31mm-5mm
<b>Working Distance</b>	<370mm
<b>Illumination- Range of Brightness</b>	>50000 lux

# SPECIFICATIONS (continued)

## MOVEMENT RANGES

Longitudinal (In/Out)	100mm
Lateral (Left/Right)	107mm
vertical (Up/Down)	30mm
Chin rest Range	80mm

## DISPOSAL

This product does not generate any environmentally hazardous residues. At the end of its product life, follow your local laws and ordinances regarding the proper disposal of this equipment.

## SOFTWARE REVISION

There is no software installed in this unit.


## SYSTEM CLASSIFICATION

- Classification of applied part by grade of protection against electric shocks: B type applied part  
B type applied part provides a certain grade of protection against electric shocks, particularly with regard to leak current, measuring current for patient, and reliability of connection to protective facility (Class I equipment).
- Type of protection against electric shocks: Class I equipment  
Class I equipment does not depend on basic insulation only for electric shocks; it also provides means for connecting the equipment to a protective grounding system so that the accessible metal parts do not become conductive in case of failure in the basic insulation.
- Grade of protection against a hazardous ingress of water: IPxO  
This product does not provide protection against ingress of water.  
(Grade of protection against a hazardous ingress of water stated by IEC 60529: IPxO)
- Classification by the method of sterilization/disinfection: None  
This product has no part requiring sterilization/disinfection.
- Classification of safety of use in an environment containing air/combustible gas, oxygen or nitrogen monoxide/combustible anesthetic gas: Equipment not suited for use in an environment containing air/combustible gas, oxygen or nitrogen monoxide/combustible anesthetic gas  
Use this product in an environment not containing combustible anesthetic gas and combustible gas.
- Classification by mode of operation: Continuous operation equipment  
Continuous operation refers to an operation under normal load conditions without exceeding specified temperatures, without a limit of time.

## RESTOCKING CHINREST TISSUE

When the chinrest tissue supply is depleted, pull out the chinrest tissue pins and replace the tissue.

# SYSTEM CONFIGURATION

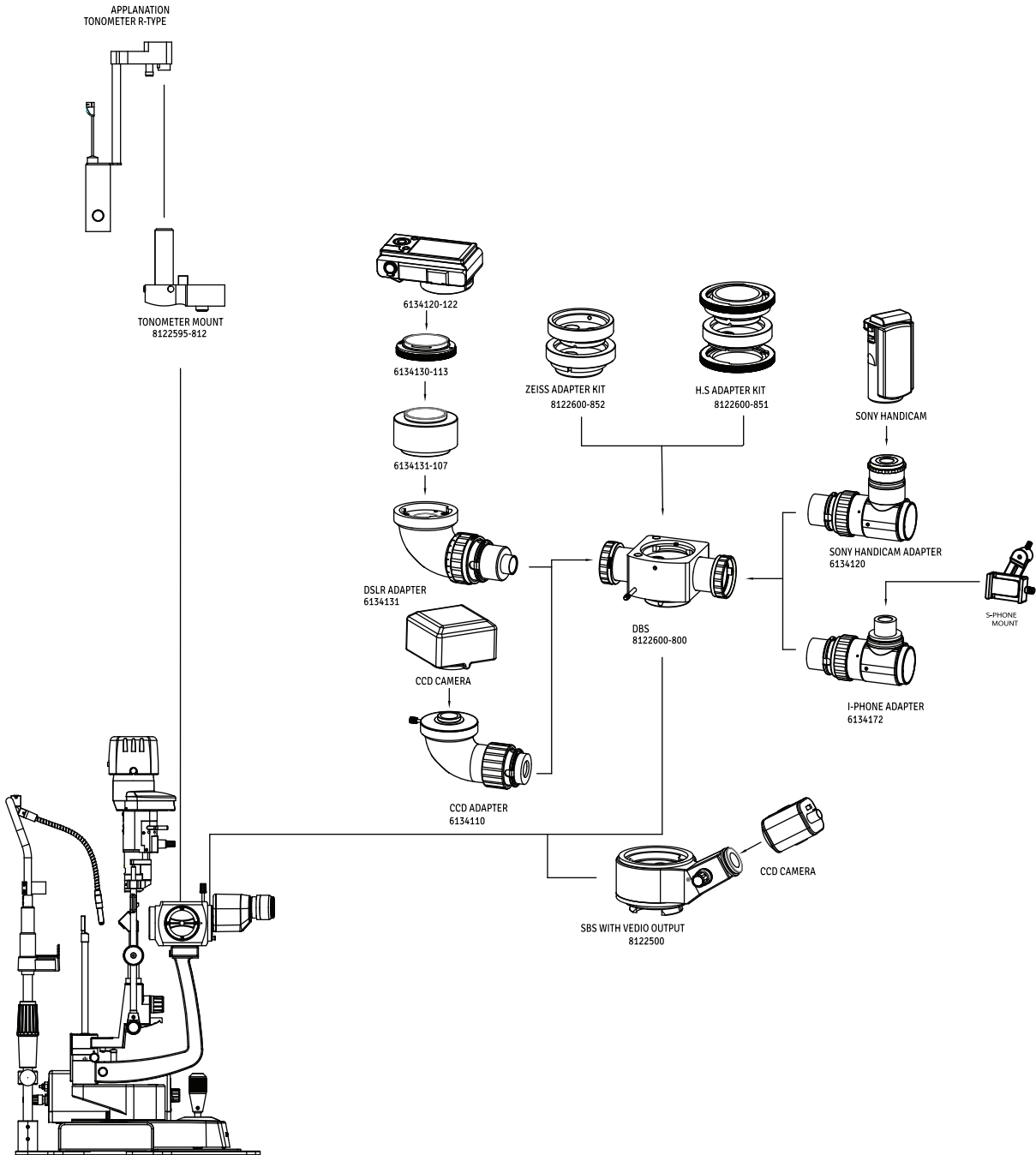


# GUIDANCE TABLES

<b>Table 201 – Guidance and Manufacturer’s Declaration</b> <b>Electromagnetic Emissions</b> <b>All Equipment and Systems</b>		
<b>Guidance and Manufacturer’s Declaration – Electromagnetic Emissions</b>		
<p>The SLx 45 is intended for use in the electromagnetic environment specified below. The customer or user of the SLx 45 should ensure that it is used in such an environment.</p>		
<b>Emissions Test</b>	<b>Compliance</b>	<b>Electromagnetic Environment</b> <b>- Guidance -</b>
RF Emissions CISPR 11	Group 1 Class A	The SLx 45 uses RF energy only for its internal function. Therefore, its RF emissions are very low and are not likely to cause any interference in near by electronic equipment.
Harmonics IEC 61000-3-2	Class A	The SLx 45 is suitable for use in all establishments, other than domestic, and those directly connected to the public low-voltage power network that supplies building used for domestic purposes.
Flicker IEC 61000-3-3	Complies	

# GUIDANCE TABLES (continued)

<b>Table 202 – Guidance and Manufacturer’s Declaration</b> <b>Electromagnetic Immunity</b> <b>All Equipment and Systems</b>			
<b>Guidance and Manufacturer’s Declaration – Electromagnetic Immunity</b>			
<p>The SLx 45 is intended for use in the electromagnetic environment specified below. The customer or user of the SLx 45 should ensure that it is used in such an environment.</p>			
<b>Immunity Test</b>	<b>IEC 60601 Test Level</b>	<b>Compliance Level</b>	<b>Electromagnetic Environment - Guidance</b>
ESD IEC 61000-4-2	±6kv Contact ±8kv Air	±6kv Contact ±8kv Air	Floors should be wood, concrete or ceramic tile. If floors are synthetic, the R/H should be at least 30%.
EFT IEC 61000-4-4	±2kv Mains ±1kv I/Os	±2kv Mains ±1kv I/Os	Main power quality should be that of a typical commercial or hospital environment.
Surge IEC 61000-4-5	±1kv Differential ±2kv Common	±1kv Differential ±2kv Common	Main power quality should be that of a typical commercial or hospital environment.
voltage Dips/Dropout IEC 61000-4-11	>95% Dip for 0.5 Cycle 60% Dip for 5 Cycles 30% Dip for 25 Cycles >95% Dip for 5 Seconds	>95% Dip for 0.5 Cycle 60% Dip for 5 Cycles 30% Dip for 25 Cycles >95% Dip for 5 Seconds	Mains power quality should be that of a typical commercial or hospital environment. If the user of the SLx 45 requires continued operation during power mains interruptions, it is recommended that the SLx 45 be powered from an uninterruptible power supply or battery.
Power Frequency 50/60Hz Magnetic Field IEC 61000-4-8	3A/m	3A/m	Power frequency magnetic field should be that of a typical commercial or hospital Environment.

# GUIDANCE TABLES (continued)

<p align="center"><b>Table 204 – Guidance and Manufacturer’s Declaration</b>  <b>Electromagnetic Immunity</b>  <b>Equipment and Systems that are NOT Life-supporting</b></p>			
<p align="center"><b>Guidance and Manufacturer’s Declaration – Electromagnetic Immunity</b></p>			
<p>The SLx 45 is intended for use in the electromagnetic environment specified below. The customer or user of the SLx 45 should ensure that it is used in such an environment.</p>			
<p align="center"><b>Immunity Test</b></p>	<p align="center"><b>IEC 60601 Test Level</b></p>	<p align="center"><b>Compliance Level</b></p>	<p align="center"><b>Electromagnetic Environment - Guidance</b></p>
<p>Conducted RF IEC 61000-4-6</p>	<p>3 vrms 150 kHz to 80 MHz</p>	<p>(v1) = 3 vrms</p>	<p>Portable and mobile Rf communications equipment should be used no closer to any part of the SLx 45, including cables, than the recommended separation distance calculated from the equation applicable to the frequency of the transmitter.</p> <p>Recommended Separation Distance:</p> <p><math>d=(3.5/v1)(\text{Sqrt } P)</math></p> <p><math>d=(3.5/E1)(\text{Sqrt } P)</math> 80 to 800 MHz</p> <p><math>d=(7/E1)(\text{Sqrt } P)</math> 800 MHz to 2.5 GHz</p> <p>Where P is the max output power rating of the transmitter in watts (W) according to the transmitter manufacturer and d is the recommended separation distance in meters (m).</p> <p>Field strengths from fixed Rf transmitters, as determined by an electromagnetic site survey, should be less than the compliance levels in each frequency range.</p> <p>Interference may occur in the vicinity of equipment marked with the following symbol.</p> 
<p>Radiated RF IEC 61000-4-3</p>	<p>80 MHz to 2.5 GHz @ 3V/m</p>	<p>(E1) = 3 v/m</p>	
<p>Note 1: At 80 MHz and 800 MHz, the higher frequency range applies.                  Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects and people.                  * Field strengths from fixed transmitters, such as base stations for radio (cellular/cordless) telephones and land mobile radios, amateur radio, AM and FM radio broadcast and Tv broadcast cannot be predicted theoretically with accuracy. To assess the electromagnetic environment due to fixed RF transmitters, an electromagnetic site survey should be considered. The measured field strength in the location in which the ME Equipment or ME System should be observed to verify normal operation. If abnormal performance is observed, additional measures may be necessary, such as re-orienting or relocating the ME Equipment or ME System.                  * Over the frequency range 150 kHz to 80 MHz, field strengths should be less than [V1] V/m.</p>			

# GUIDANCE TABLES (continued)

**Table 206 – Recommended Separation Distances between Portable and Mobile RF Communications Equipment and the SLx 45 for ME Equipment and ME Systems that are NOT Life-supporting.**

## **Guidance and Manufacturer’s Declaration - Electromagnetic Immunity**

### **Recommended Separation Distances between Portable and Mobile RF Communications Equipment and the Xpert SL- 45**

The Xpert SL-45 is intended for use in the electromagnetic environment in which radiated RF disturbance are controlled. The customer or user of the Xpert SL-45 can help prevent electromagnetic interference by maintaining a minimum distance between portable and mobile RF Communications Equipment (transmitters) and the Xpert SL-45 as recommended below, according to the maximum output power of the communications equipment.

<b>Max Output Power of Transmitter (W)</b>	<b>Separation (m) 150kHz to 80 MHz</b> $d=(3.5/v1)(\text{Sqrt } P)$	<b>Separation (m) 80 to 800 MHz</b> $d=(3.5/E1)(\text{Sqrt } P)$	<b>Separation (m) 800MHz to 2.5GHz</b> $d=(7/E1)(\text{Sqrt } P)$
0.01	0.1166	0.1166	0.2333
0.1	0.3689	0.3689	0.7378
1	1.1666	1.1666	2.3333
10	3.6893	3.6893	7.3786
100	11.6666	11.6666	23.3333

For transmitters rated at a maximum output power not listed above, the recommended separation distance (d) in meters (m) can be estimated using the equation applicable to the frequency of the transmitter, where P is the maximum output power rating of the transmitter in watts (w) according to the transmitter manufacturer.  
 Note 1: At 80 MHz and 800 MHz, the separation distance for the higher frequency range applies.  
 Note 2: These guidelines may not apply in all situations. Electromagnetic propagation is affected by absorption and reflection from structures, objects, and people.



## WARRANTY

This product is warranted by Labo America Inc. against defective material and workmanship under normal use for a period of one year from the date of invoice to the original purchaser. (An authorized dealer shall not be considered an original purchaser). Under this warranty, Labo America Inc. sole obligation is to repair or replace the defective part or product at Labotech/Labomed discretion.

This warranty applies to new product and does not apply to a product that has been tampered with, altered in any way, misused, damaged by accident or negligence, or which has had the serial number removed, altered or effaced. Nor shall this warranty be extended to a product installed or operated in a manner not in accordance with the applicable LABOMED instruction manual, nor to a product which has been sold, Serviced, installed or repaired other than by a Labo America Inc. factory or authorized LABOMED Dealer.

Lamps, bulbs, charts, cards and other expendable items are not covered by this warranty.

All claims under this warranty must be in writing and directed to the LABOMED factory, or authorized instrument dealer making the original sale and must be accompanied by a copy of the purchaser's invoice.

This warranty is in lieu of all other warranties implied or expressed. All implied warranties of merchantability or fitness for a particular use are hereby disclaimed. No representative or other person is authorized to make any other obligations for a LABOMED product. Labotech/Labomed shall not be liable for any special, incidental, or consequent damages for any negligence, breach of warranty, strict liability or any other damages resulting from or relating to design, manufacture, sale, use or handling of the product.

### PATENT WARRANTY

If notified promptly on writing of any action brought against the purchaser based on a claim that the instrument infringes a U.S. Patent, Labo America Inc. will defend such action at its expense and will pay costs and damages awarded in any such action, provided that Labo America Inc. shall have sole control of the defense of any such action with information and assistance (at Labo America Inc. expense) for such defense, and of all negotiation for the settlement and compromise thereof.

### PRODUCT CHANGES

Labo America Inc. reserve the right to make changes in design or to make additions to or improvements in its products without obligation to add such to product previously manufactured.

### CLAIMS FOR SHORTAGES

We use extreme care in selection, checking, rechecking and packing to eliminate the possibility of error. If any shipping errors are discovered:

1. Carefully go through the packing material to be sure nothing was inadvertently overlooked when the unit was unpacked.
2. Call the dealer you purchased the product from and report the shortage. The material are packed at the factory and none should be missing if the box has never been opened.
3. Claims must be filed within 30 days of purchase.

### CLAIMS FOR DAMAGES IN TRANSIT

Our shipping responsibility ceases with the safe delivery in good condition to the transportation company. Claims for loss or damage in transit should be made promptly and directly to the transportation company. If, upon delivery, the outside of the packing case shows evidence of rough handling or damage, the transportation company's agent should be requested to make a "Received in Bad Order" notation on the delivery receipt. If within 48 hours of delivery, concealed damage is noted upon unpacking the shipment and no exterior evidence of rough handling is apparent, the transportation company should be requested to make out a "Bad Order" report. This procedure is necessary in order for the dealer to maintain the right of recovery from the carrier.

## Revision History

Rev. No.	Date of Release	DCR #	Change	App. By
1.4	April 4,2019	DCR/11/19	System Diagram Added	S Bal
1.5	June 29,2020	DCR/14/20	Cleaning and disinfection	S Bal
1.6	Sep,2022	DCR/15/22	Iphone Adapter Replaced With Universal adaptor	S Bal
1.7	March 29,2023	ECR/07/23	Electrical Specification Change	S Bal



**Labo America Inc.**  
920 Auburn Court  
Fremont, CA  
94538  
U.S.A.

Phone: 510-445-1257

Fax: 510-991-9862

Email: [sales@laboamerica.com](mailto:sales@laboamerica.com)  
[www.laboamerica.com](http://www.laboamerica.com)

**EU|REP.**

**Labomed Europe**  
Essebaan 50  
NL-2908 LK Capelle a/d IJssel  
The Netherlands  
Tel: +31 (0)10 4584222  
Fax: +31 (0)10 4508251  
E-mail: [info@labomedeuropa.com](mailto:info@labomedeuropa.com)



DQS Inc.  
Labo America Inc.  
Registered to ISO 9001:2015  
BR Number 10005565



LABOTECH MICROSCOPES INDIA Pvt Ltd